

Parts

When ordering please use the part numbers shown on the images below.



Online: www.eramask.com







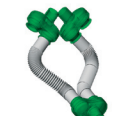




Email: info@eramask.com

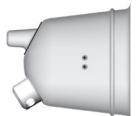

By phone: (+61 3) 9553 7200
Free call in Australia: 1800 006 956



By Fax: (+61 3) 9553 7299

	Mask Shell		MDI Spacer
	Seal		1.5 metre Strap with Quick Release Cord
	Non-Rebreathing Valves and Connector Assembly		Nebuliser Pump High Flow 230V
	Wet Nebuliser Bottle 6cc		Nebuliser Pump High Flow 115V
	Wet Nebuliser Kit		

Caring for your ERA Mask

To clean the mask disconnect accessories, valves and connector assembly. Soak the parts and the shell in warm water with a mild detergent for 10-20 minutes. Drain and shake off excess water and allow to air dry. Do not heat sterilize the mask.

Check the seal and valves before use and replace if worn or damaged.

Store in a dry, well ventilated area away from direct sunlight and extreme temperatures.

DISTRIBUTED BY

INTERNATIONAL HEAD OFFICE
BIOMEDTECH Australia Pty Ltd
17 Roberna Street, Moorabbin, Victoria, 3189
Ph: (03) 9553 7200 Fax: (03) 9553 7299
Freecall: 1800 006 956
Email: info@eramask.com
Web: www.eramask.com

ERA® Mask

A NEW ERA IN EQUINE RESPIRATORY TREATMENT




www.eramask.com

- Perfect and soft seal
- Safe quick release
- Small dead space
- Lightweight and rigid shell
- Versatile accessories
- Comfortable low resistance breathing
- Registered design, external breathing valves

IHT Package

The IHT ERA Mask package has been developed for use with GO₂Altitude® Hypoxicators, a new drug free performance enhancing technology. The soft seal has been specifically designed to ensure that there is minimal leakage of ambient air into the mask during a simulated altitude training session. The minimal dead space eliminates carbon dioxide build up in the breathing circuit.

170262

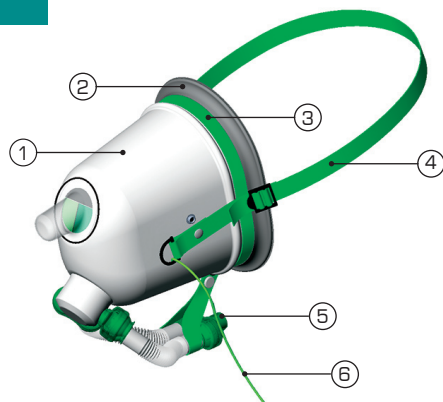


1. Mask Shell
2. Seal
3. Strap Hose Support
4. Strap 1.5 m Adjustable
5. Non-Rebreathing Valves and Connector Assembly
6. Quick Release Cord

* spare seal and quality carry bag included in the package

How to Use ERA Mask with GO₂Altitude®

1. Please read the safety information carefully, before using ERA mask.
2. Place the mask onto the horse's muzzle.
3. Secure the strap behind the horse's ears and adjust if necessary.
4. Once the horse is comfortable with the mask, connect the hose from the GO₂Altitude hypoxicator onto the tube connector ⑤.

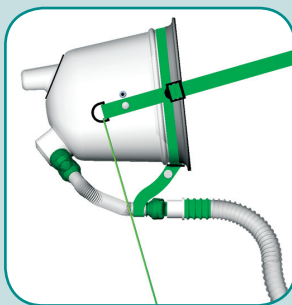


For more information on Equine Intermittent Hypoxic Training (IHT) please visit our web site <http://www.go2altitude.com/horse.html>

Veterinary Package

The ERA Mask veterinary package provides a system to efficiently administer medication to the horse via a Metered Dose Inhaler (MDI) or Wet (Aerosol) Nebuliser. The excellent seal, small dead space and registered design non-rebreathing valves ensure that the horse gets the maximum benefit from the medication provided.

170372

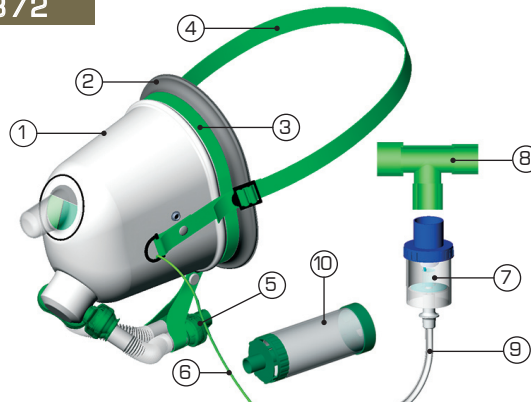


1. Mask Shell
2. Seal
3. Strap Hose Support
4. Strap 1.5 m Adjustable
5. Non-Rebreathing Valves and Connector Assembly
6. Quick Release Cord
7. Wet Nebuliser Bottle
8. 3-way Connector
9. Tubing 1.8 m
10. MDI Spacer

* spare seal and quality carry bag included in the package

How to Use ERA Mask with an MDI

1. Please read the safety information carefully, before using ERA mask.
2. Place the mask onto the horse's muzzle.
3. Secure the strap behind the horse's ears and adjust if necessary.
4. Prepare the MDI device as per its manufacturer's or veterinarian's instructions and insert into the MDI spacer ⑩.
5. Once the horse is comfortable plug the MDI Spacer ⑩ into the tube connector ⑤.
6. Operate the MDI as per the drugs usage or veterinarian's instructions.

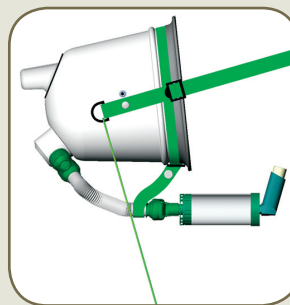

SAFETY

1. Before administering any respiratory treatments for your horse please consult your veterinarian.
2. To ensure minimal distress to your horse fit the mask without any accessories attached. Once the horse is comfortable with the mask, accessories can be fitted to the connector ⑤. Some horses will not be comfortable with wearing the mask.
3. Always hold the quick release cord in case the horse becomes agitated and the mask needs to be removed quickly. If required, pull the cord ⑥ away from the mask, releasing the strap. The mask can now be removed or will be shaken off by the horse.
4. Refer to the information provided with the medication for dosage and correct application. If in doubt consult your veterinarian.
5. ERA Mask is intended for single horse use to prevent cross-contamination.

How to Use ERA Mask with a Wet Nebuliser

1. Please read the safety information carefully, before using ERA mask.
2. Place the mask onto the horse's muzzle.
3. Secure the strap behind the horse's ears and adjust if necessary.
4. Prepare the Nebuliser bottle ⑦ with dosage as per the drugs usage or veterinarian's instructions.
5. Connect the tube ⑨ from the pump and plug the bottle ⑦ into the 3-way connector ⑧.
6. Once the horse is comfortable, plug the assembled Nebuliser onto the tube connector ⑤.
7. Administer the medication as per the drugs usage or veterinarian's instructions.

